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Instructions for use by patients

Bluemix bleaching gel is intended for brightening discoloured teeth. **In accordance with instructions given at your dentist's surgery (office)**, the gel is filled at home into the individually produced flexible tray and is supposed to take effect on the teeth for 2–4 hours.

Composition

Carbamide peroxide, glycerine, buffered polycarboxylic acid, e.g. banana aroma, artificial sweetener.

General remarks and contra-indications

- Use only for brightening teeth.
- Fillings and dentures are not brightened.
- **Store** in a cool, dark place **at 2–8 °C**, but do not freeze.
- Do not overfill the tray, avoid contact with skin and mucous membrane.
- Not to be used in the event of an allergy to the constituents.
- Do not use on pregnant women and during breast-feeding.
- Do not use on children, heavy smokers and in the case of high alcohol consumption.
- Do not use force to press the gel out of the syringe.

Side-effects

In individual cases, irritations of the mucous membranes and sensitive teeth may result. Should the symptoms not disappear after the interruption of treatment, you should consult your dentist.

Warning remark!

The component carbamide peroxide has an irritant effect. In the event of contact with eyes and mucous membranes, rinse with lots of water and if necessary consult a doctor.

Do not eat or smoke during brightening treatment.

This is how it is done

- Before brightening, brush your teeth and clean the interstices.
- When using for the first time, break off the cap of the plastic syringe and keep it for later closing. **(1)**
- Carefully apply small quantities of gel (0.5 ml per tray!) to the inside of the trays facing the outside face of the tooth. **(2)**
- Close the syringe again with the cap previously broken off. **(3)**
- Insert the trays without any pressure.
- Remove any surplus with a toothbrush or finger. Do not swallow it!
- **Length of time trays are to be worn 2–4 hours per day.**
- After that thoroughly brush trays and teeth and rinse with water. **(4)**
- Place cleaned trays in the storage box and store the syringes in a cool place.
- After four days, the first results can be seen. After 7 days, the treatment has been concluded.

Keep out of reach of children!

Only intended for use in the oral cavity!

Application only with specialist dental advice!

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